FAST Tactical

Sternal Intraosseous Device

FOR STERNAL USE ONLY

For use only on patients 12 years and older

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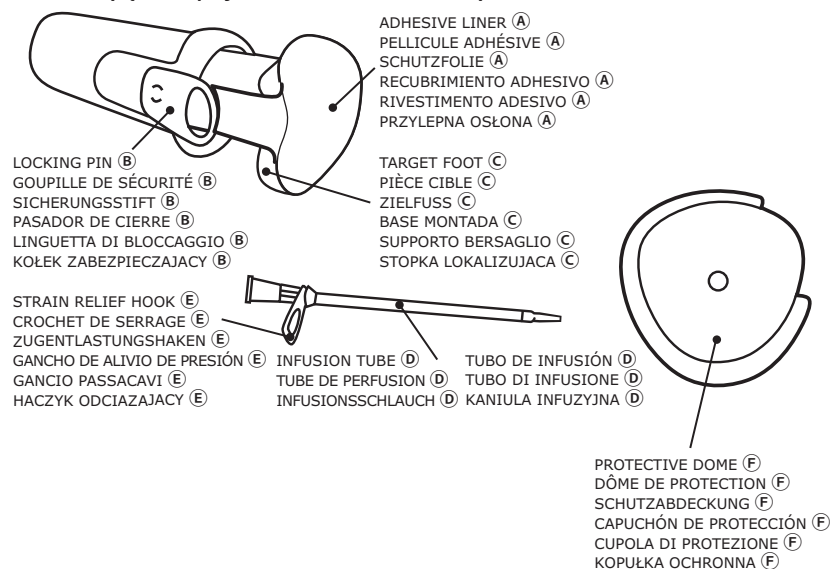
EN CONTENTS: Sternal Intraosseous Device.

FASTTactical™ Sternal Intraosseous Device is intended for intraosseous infusion as an alternative to intravenous access to facilitate emergency resuscitation through the use of drugs and fluids.

The **FASTTactical™** Sternal Intraosseous Device is indicated for use in establishing a sternal intraosseous access route in adult and adolescent patients (12 years of age and older), requiring vascular administration of drugs or fluids to facilitate emergency resuscitation.

The **FASTTactical™** is not intended to be left in situ for more than 24 hours

DESIGNATED INSERTION SITE: manubrium, on the midline 15mm below the supra-sternal notch

CONTRAINDICATIONS: none known

PRECAUTIONS:

- The **FASTTactical™** is designed to penetrate 6mm into the manubrium. Qualified professionals should determine any appropriate or necessary exceptions, either inclusions or exclusions, to the criterion "for patients 12 years and older".
- Proximal tip of Infusion Tube contains metal.

The function of the device may be affected by:

- Compromised skin over the insertion site such as trauma, infection or burns
- Fracture of the sternum or vascular injury which may compromise the integrity of the manubrium or its vascularization
- Midline sternotomy scars

WARNINGS:

- Safety in patients with very severe osteoporosis has not been proven
- Insertion in sites other than the manubrium may result in ineffective infusion and/or serious injury to the patient
- Reuse of **FASTTactical™** is not recommended due to the potential of cross-contamination, which may lead to serious injury or death. The **FASTTactical™** is unlikely to function after use
- Do not insert finger(s) in the open end of the device due to the potential of needle stick.

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
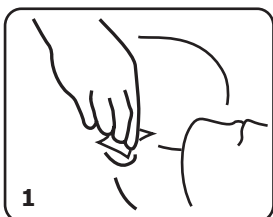
Authorized Representative in the European Community

Emergo Europe
Molenstraat 15,
2513 BH The Hague
The Netherlands
+31 (0) 70 345 8570

EN 1. Expose the sternum and clean infusion site. **Note:** maintain aseptic technique throughout the procedure. Note: administer local anaesthetic if the patient is conscious and alert.

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
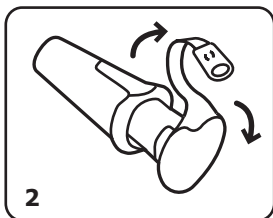
EN 3. Align the Target Foot (C) notch with the patient's sternal notch, over the midline, and perpendicular to the manubrium.

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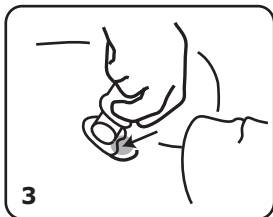
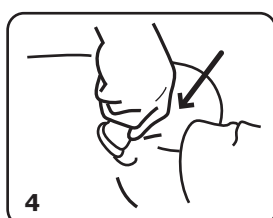
EN 5. Withdraw the FASTTactical™ straight back while holding down the Target Foot (C). Support comes out with the Infusion Tube. Discard the FASTTactical™ following local contaminated sharps protocols.

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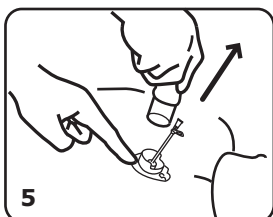
EN 6. Connect the IV line directly to the luer, and clip the Strain Relief Hook (E) to the Target Foot (C). *Optional:* confirm placement by aspiration, flush with fluid to clear.

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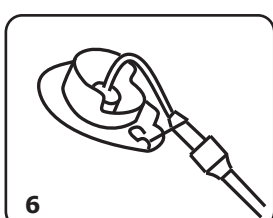EN 7. *Optional:* Remove the liner from the Protective Dome (F) and apply the Dome (F) over the Target Foot (C) infusion site.

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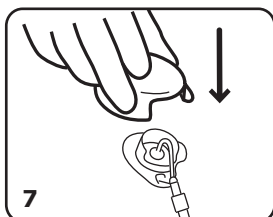EN Notes: If fluid does not flow, even after flushing, or if extravasation occurs, infusion should be discontinued and an alternate method of vascular access be used.

Fluids or drugs may be administered as boluses from a syringe, or from fluid sources using gravity drip, pressure cuffs or syringe/stopcock pumping method. Fluids have been infused into sternums at 30 ml/min by 1m gravity drip and 120 ml/min by pressurized source.

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